



Directives for the photography and evaluation of the print-out

Since the colour composition of the ArtificialIris is custom-made on the basis of a photo print-out utmost care should be taken to receive the highest quality photographs and print-outs (hard copies) for each eye.


Please notice the following details:

- Light the patient's eye evenly, without shadows to the iris.
To avoid glares adjust the light sources from two sides on the object.
- Before taking the photo, a white balance of the camera is necessary (see instruction manual of the digital camera). Therefore, instead of the patient's eye a white paper or a grey chart may be used. Please do not displace the adjusted lighting.
- After that place the patient in the same position and take a photo of each eye.
Do not use a video camera for photography.

Examples:



too far away

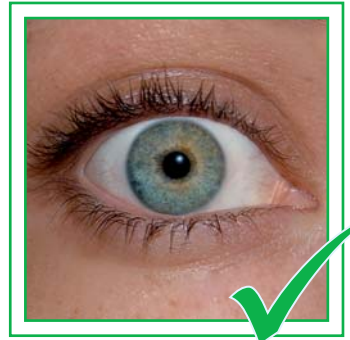

low lid, too blurry

- **Use** a photo printer with special photo paper to print-out your digital images.



If you are not set up for high quality photography and photo print-outs we recommend to **send the patient to a professional photographer** of their choice to provide the patient and doctor with the high quality photo and print-out of each eye needed for their approval as the target colour for production.



Evaluate your photo print-out (**HARD COPY**):

- Check the colour of the conjunctiva on the print. If it isn't white the picture has a colour deviation.
- Compare the colour of the photo with the natural iris (both, photo and natural iris, under the same illumination).



Only you and your patient on location can evaluate and select the photo print-out most closely matching his/her natural iris.

- ➔ Please **DO NOT** send the photos **DIGITALLY** via email only because we cannot make sure that the appearance of the digital image on our screen is identical to the colour composition of the patient's natural iris, and we also cannot verify the colour match to the natural iris when we print out the digital photos ourselves.
- ➔ Please let your patient sign the order form agreeing with the photo selection you make before sending us the labelled pictures together with the appropriately filled-in order form by **mail** to:

Dr. Schmidt Intraocularlinsen GmbH
Westerwaldstr.11-13, 53757 St. Augustin, Germany